RapidFill™ ASF Syringe Strip

Ruban de seringues ASF RapidFill • RapidFill ASF Spritzenplaster • Cinta de Jeringas para Sistema de Relleno Automatizado RapidFill ASF • Striscia di siringhe per Sistema di Riempimento Automatico per Siringhe (ASF) RapidFill

Sterile 10 mL syringes with twist-off luer tip caps banded with label material. For use with RapidFill Automated Syringe Filler (ASF).

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Specifications

Spécifications • Spezifikationen • Especificaciones • Specificazioni

Verify compatibility of syringe materials of construction with fluids to be filled.

- Tip Cap: thermoplastic elastomer
- Syringe:
 - Barrel – polypropylene
 - Plunger – polypropylene
 - Plunger Tip – thermoplastic elastomer
- Webbing:
 - Clear Film – polypropylene
 - Label – polypropylene

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Precaution:

Avoid direct exposure of cleaning solutions to syringe label. These solutions (i.e., Isopropyl alcohol, iodine, bleach, etc.) might cause discoloration/damage to syringe label or damage label text.

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Contraindications:

None when used as described in product insert.

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fr

1. Raise syringe cover.
2. Open top flap of syringe strip box and secure into place by engaging the snap tab on the box top.
3. Position plastic syringe strip box in front of Automated Syringe Filler and slide onto syringe handling module. (Figure 1)
4. Locate beginning of syringe strip.
5. Using left hand, place first syringe (beginning of strip) in ASF, while simultaneously pressing down the syringe-loading lever with right hand. (Figure 2)
6. After first syringe is properly positioned, release syringe-loading lever.
7. Lower syringe cover.

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Figure 1

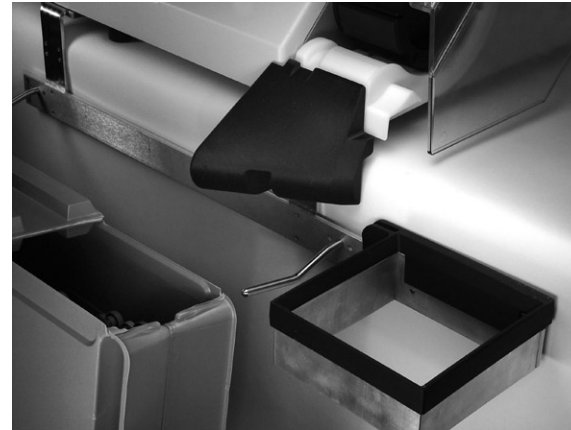
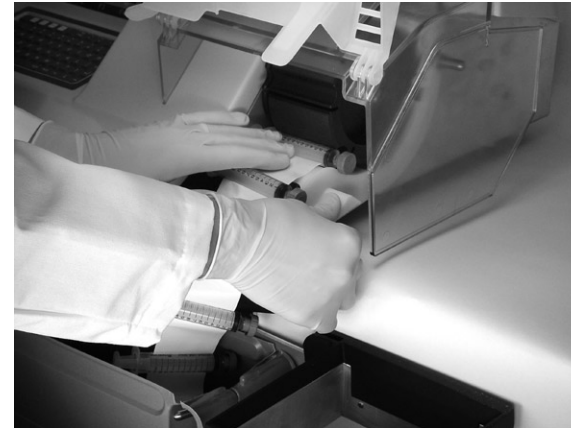
Figure 2

Note: Barcodes printed on the Fluorescent Red label Syringe Strip may not be able to be read by all barcode scanners.

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RapidFill is a trademark of Baxa Corporation.

Baxa and the Baxa logo are registered trademarks of Baxa Corporation.

CE
0086

ISO
13485
CERTIFIED
QUALITY SYSTEM

Baxa

Corporate Headquarters USA - Baxa Corporation
9540 S. Maroon Circle, Suite 400 | Englewood, CO 80112
tel: 303-690-4204 | fax: 303-690-4804
toll free: 800-567-BAXA (2292) | e-mail: baxamail@baxa.com

Registered Agent United Kingdom - Baxa Ltd.
Radius Court, Eastern Road | Bracknell, RG12 2UP
tel: +44 (0)1344 392902 | fax: +44 (0)1344 392903
email: info@baxa.co.uk | www.baxa.com

Systems Development Center
790 Fentress Blvd. | Daytona Beach, FL 32114
tel: 386-274-4447 | fax: 386-274-4114
toll free: 888-434-8462 | email: supplies@fhtinc.com

To find a local Baxa agent, visit our Web site at www.baxa.com or call our offices directly.